

No Sting Barrier Film Swab and Wipe

Instructions for Use

1.0 Product Description

Halyard Barrier Film is a polymeric solution which forms a uniform film when applied to the skin using a swab (foam applicator) or wipe. The product is dispersed in a non-stinging solvent, which dries rapidly. The film is colourless, transparent, and possesses good oxygen and moisture vapour permeability. **Halyard** Barrier Film helps to protect intact or damaged skin from irritation caused by urine and/or fecal incontinence, digestive juices, wound drainage.

2.0 INTENDED PURPOSE

Halyard Barrier Film is a liquid intended for use as a film-forming product that upon application to intact or damaged skin forms a long lasting waterproof barrier, which acts as a protective interface between the skin and bodily wastes, fluids. It is intended as a primary barrier against irritation from body fluids and can be used on adults and children.

2.1 INTENDED POPULATION

The product may be used on adults, children and infants over 12 weeks old.

2.2 INTENDED USERS

Intended for use by health professionals and may be used in a hospital, community and home setting.

3.0 INDICATIONS

The products are indicated for protecting intact or damaged skin from body fluids in the areas of:

- Incontinence skin care
- Peri-stomal skin protection
- Peri-wound skin protection

4.0 CONTRAINDICATIONS

Halyard Barrier Film is not to be used as the only covering in situations that require dressing protection from bacterial contamination / penetration e.g. intravenous therapy catheter sites and full or partial thickness wounds. **Halyard** Barrier Film is not to be used on infected skin.

5.0 WARNING



Halyard Barrier Film, in liquid form, is flammable; use in a well ventilated area. Avoid using around flames and sources of ignition. Keep out of reach of children.



Do not use if pack is damaged or open.

6.0 INFORMATION FOR USE

- Skin should be clean and dry prior to application of Halyard Barrier Film.
- Remove foam applicator from sachet and apply a uniform coating of film over entire area of concern.
- Visually inspect Sachet for damage prior to opening
- Follow local protocols on the handling of sterile product.
- If an area is missed, reapply to that area only after first application of Halyard Barrier Film has dried (approximately 30 seconds).
- If Halyard Barrier Film is applied to area with skin folds or other skin-to-skin contact, make sure that skin-contact areas are separated to allow the film to thoroughly dry before returning to normal position.
- When used under adhesive tapes, dressings, or devices:
 - (a) Allow Halyard Barrier Film to thoroughly dry before covering with dressing or adhesive products.
 - (b) Reapplication of Halyard Barrier Film is necessary each time dressing and/or adhesive products are changed; the barrier film is removed by the adhesive.
- When used as a protectant against body fluids, feces, or urine etc., and no adhesive products are applied to skin:
 - (a) Reapplication of Halyard Barrier Film is recommended every 24-72 hours, depending on frequency of cleaning.
 - (b) In extreme cases (e.g. constant diarrheal stooling) with very frequent cleansing, more frequent applications may be necessary (i.e. every 12-24 hours).
- If desired, the film can be removed by using most medical adhesive removers as directed. Clean and dry the involved area and reapply Halyard Barrier Film.

Ingredients

- Disiloxane, Acrylate Copolymer

7.0 CLINICAL BENEFITS /SPECIAL NOTES

Barrier films offer protection to the skin against subsequent moisture contact. The Barrier film forms a thin layer, repelling moisture and potential irritants, thus protecting the skin.

Caution

1. Should redness or any other signs of irritation appear, or if redness or signs of irritation persist, cease the use of Halyard Barrier Film and consult a physician.
2. Use of other barrier products, ointments, creams or lotions may significantly reduce the effectiveness of Halyard Barrier Film.

8.0 STORAGE







Store away from direct sunlight at ambient temperature and humidity. Store in a dry place.

9.0 DISPOSAL

This must be disposed of in compliance with anti-pollution and other laws of the country concerned. Contact your local service providers for further information

The Halyard Barrier Film Range can be disposed of based on the hospital or healthcare professional advice.

10.0 SYMBOLS ON LABELLING

WARNINGS	
	Flammable
	Do not use if the pouch is damaged or opened.
STORAGE	
	
STERILE	
	 

11.0 PRESENTATION

Halyard Barrier Film is available in the following formats:

Cat. No.	Size	No. /Box
MEDLO50015	0.75ml No Sting Barrier Film Wipe	25
MEDLO50025	1ml No Sting Barrier Film Wipe	25
MEDLO50125	1ml No Sting Barrier Film Swab	5
MEDLO50225	3ml No Sting Barrier Film Swab	5

12. COMPLAINTS

In USA, please call 1-844-425-9273 • halyardhealth.com



Manufactured for Owens & Minor Distribution, Inc., 9120 Lockwood Boulevard, Mechanicsville, VA 23116, USA

*Registered Trademark or Trademark of O&M Halyard, Inc., or its affiliates. © 2021. All rights reserved.

Made in Ireland